

Zygomatic²

PLUS



Zygomatic²

PLUS

- **AGGRESSIVE AND COMPRESSIVE THREADS**

Unique performance and faster implant locking.

- **16° MORSE TAPER CONNECTION**

High mechanical strength and better fit and adaptation of prosthetic components with a 16° internal angulation.

- **SMOOTH APICAL END**

Better adaptation and protection of soft tissues, leading to superior results in severe maxillary resorption cases.

- **MADE OF GRADE IV COLD-WORKED TITANIUM**

A very lightweight metal with high resistance to corrosion, wear, and fracture, featuring Double Acid Etching (DAE) surface treatment.

- **EXCLUSIVE HANANO PLUS SURFACE**

Highly hydrophilic, the Plus Surface promotes the osteogenesis process, stimulates bone tissue growth and early healing, and enhances implant strength and anchorage.



IIDEAL FOR IMMEDIATE LOAD



Immediate results and shorter treatment duration

The Zygomatic Plus implant provides an immediate solution for the patient, as an alternative to bone grafting procedures. It offers stability, predictability, and precision in treatment, meeting the expectations of professionals with its versatile design and options tailored to anatomical features and tissue relationships.




TECHNICAL MEASUREMENTS			
CODE	THREAD DIAMETER (MM)	PLATFORM DIAMETER (MM)	LENGTH (MM)
ILMZ 4030N	Ø4,0	Ø4,0	30,0
ILMZ 4032N	Ø4,0	Ø4,0	32,5
ILMZ 4035N	Ø4,0	Ø4,0	35,0
ILMZ 4037N	Ø4,0	Ø4,0	37,5
ILMZ 4040N	Ø4,0	Ø4,0	40,0
ILMZ 4042N	Ø4,0	Ø4,0	42,5
ILMZ 4045N	Ø4,0	Ø4,0	45,0
ILMZ 4047N	Ø4,0	Ø4,0	47,5
ILMZ 4050N	Ø4,0	Ø4,0	50,0
ILMZ 4052N	Ø4,0	Ø4,0	52,5
ILMZ 4055N	Ø4,0	Ø4,0	55,0
ILMZ 4057N	Ø4,0	Ø4,0	57,5
ILMZ 4060N	Ø4,0	Ø4,0	60,0
ILMZ 4062N	Ø4,0	Ø4,0	62,5

PROSTHETIC COMPONENTS





The multi-unit abutments with angulations of 17, 30, 45, 52, and 60 degrees and heights ranging from 2.0 mm to 4.0 mm, made of titanium alloy and anodized gold color. Its installation torque is 20 Ncm.

Protectors are screwed onto the multi-unit abutment with standard manual torque and must remain in position until the provisional or definitive prosthesis is fabricated.

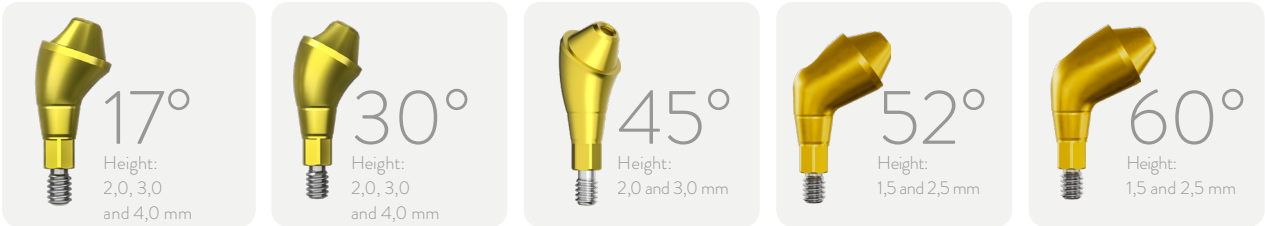

ZYGOMATIC SURGICAL KIT

Achieve superior results with high performance through a surgical protocol designed to overcome the challenges of clinical procedures.

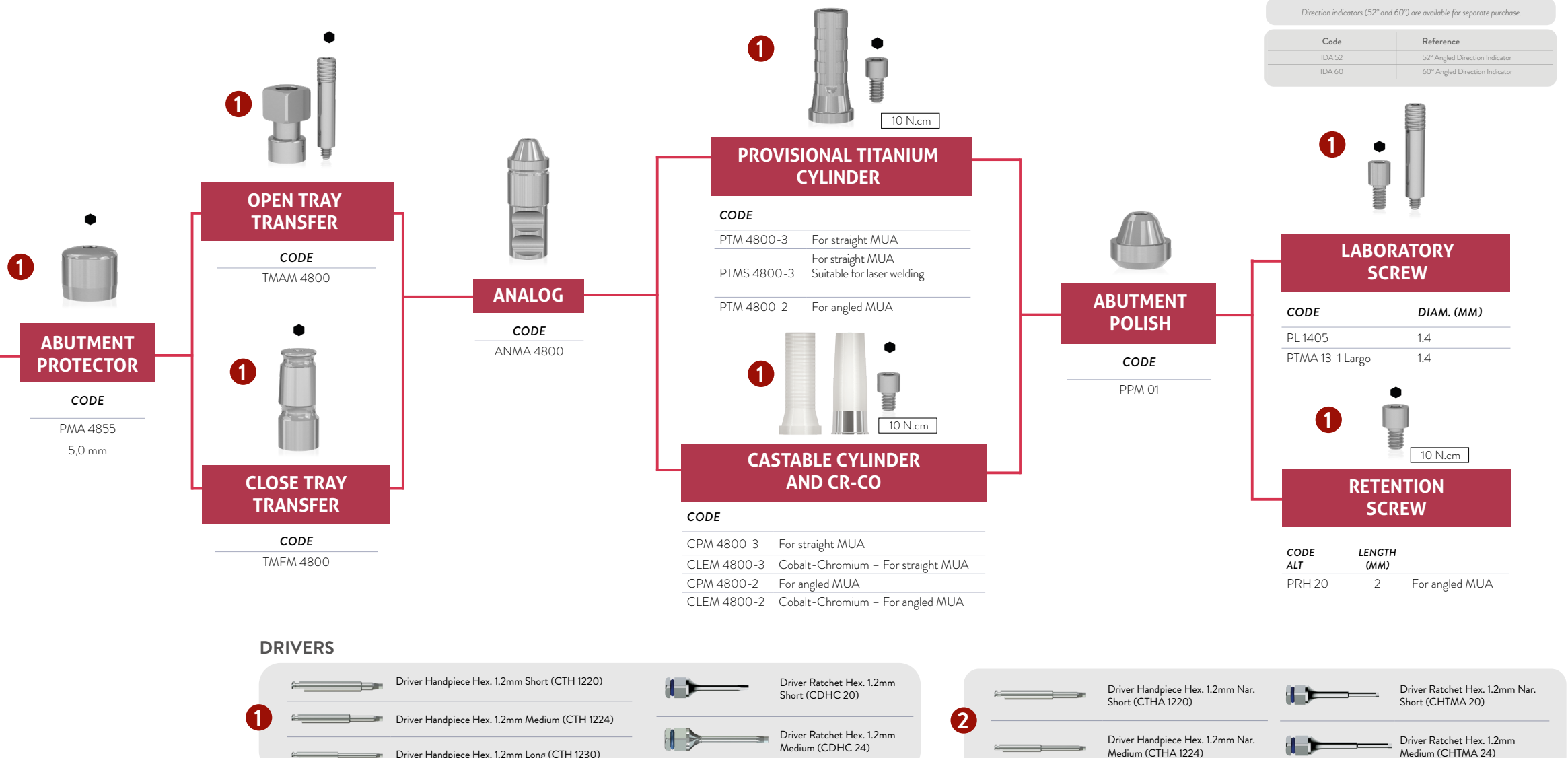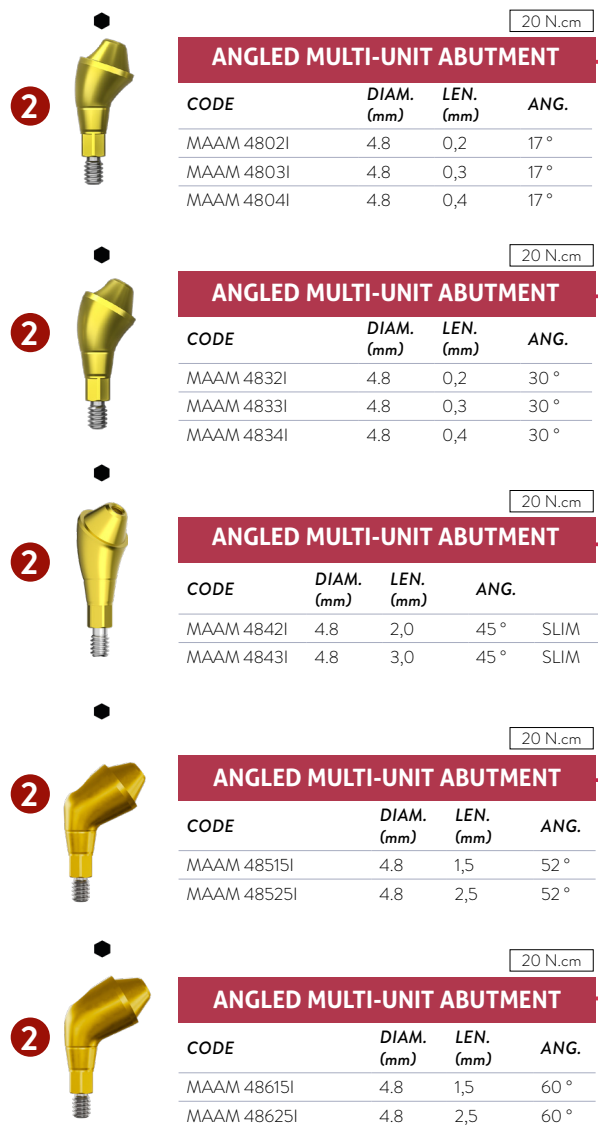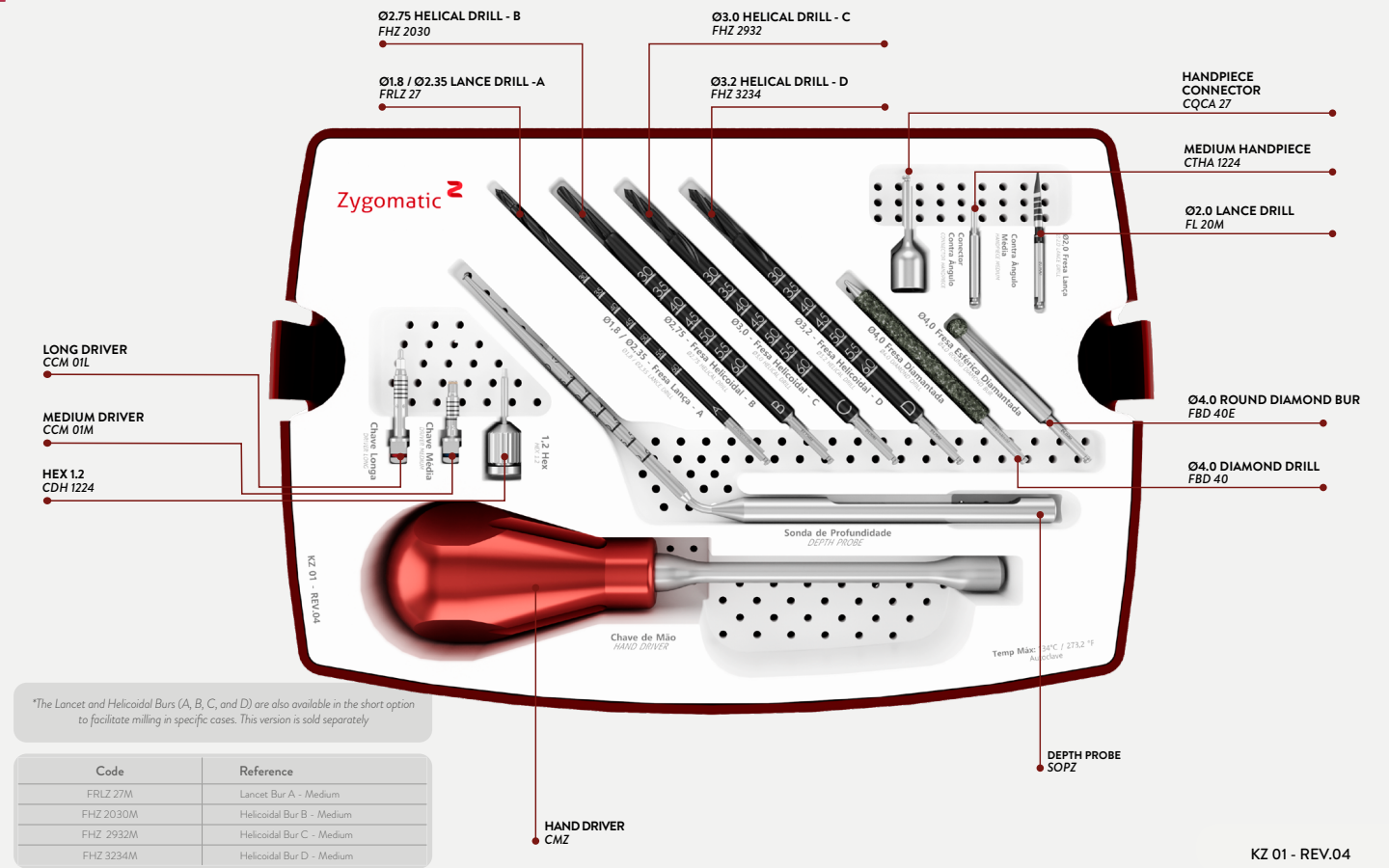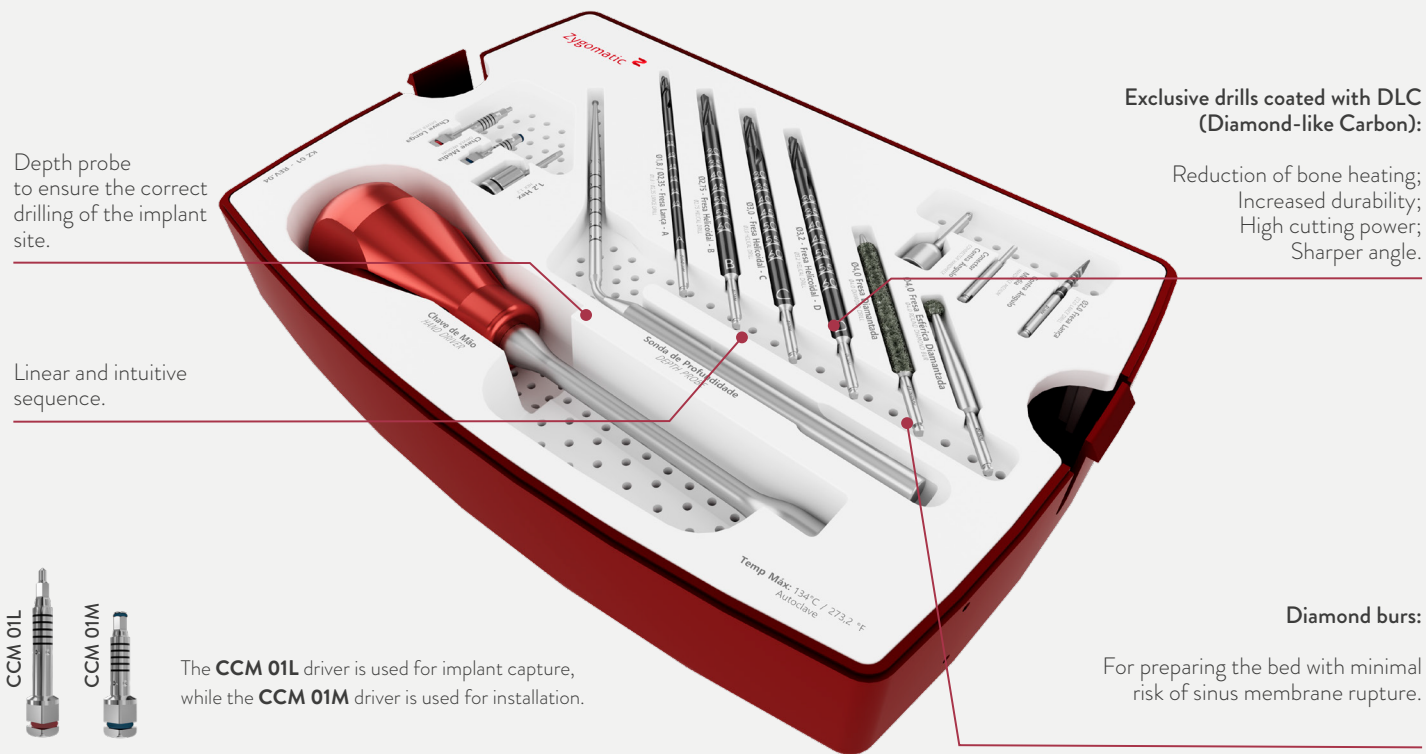



internacional@sinimplantsystem.com
www.sinimplantsystem.com

Visit our social media



@sinimplantglobal



@sinimplantglobal



S.I.N. Implant System



/sin_implant